


Please refer to the **Hologic ThinPrep LBC Pap Sample Collection Instructions**
**Please note expiry date on vial*

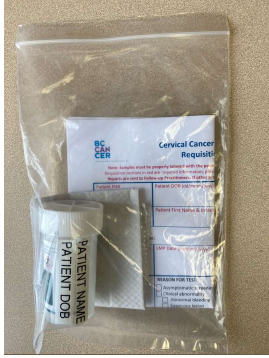

Ensure cap on specimen vial is tightly closed

Place sample vial in sealable plastic bag (one sample per bag only) + absorbant pad

- Absorbant pad examples:
- paper towel
 - gauze square
 - cotton pad

Fold completed, associated CCSL requisition and place in outer pouch in bag

Keep sample vial at 15-30°C

Shipping qualifies for "Excepted Quantities" as long as each inner bag contains 1 vial (20ml) and no more than 25 vials per outer package (max 500ml)

Place up to 25 individual bags in outer packaging for delivery to CCSL

- Outer package examples:
- padded envelope
 - cardboard box
 - other as per courier pickup

Select transport method as established

Life Labs

MTS

Other local medical courier

Canada Post Business Reply Mail
(pre-paid envelope available thru LBC kit ordering system)

Deliver to
CCSL , Central Processing & Receiving
655 W12th Avenue
Vancouver, BC V5Z 4R4

Affix label, or record on package:
Exempt Human Specimen