

LBC SAMPLES for HPV and/or cytology

Please refer to the **Hologic ThinPrep LBC Pap Sample Collection Instructions**
**Please note expiry date on vial*

Ensure cap on specimen vial is tightly closed
by lining up black lines on vial and lid



Place sample vial in
sealable plastic bag
(one sample per bag only)
+
absorbant pad

Absorbant pad examples:

- paper towel
- gauze square
- cotton pad

Fold completed associated CCSL
requisition and place in bag

Keep sample vial at room temp (15-30°C)

Place up to 25 individual bags in outer
packaging and mark for delivery to CCSL

Outer package examples:

- envelope, bag
- cardboard box
- other as per courier pickup

LABEL OUTER PACKAGE
“CCSL” with address (below)

Shipping qualifies for
“Excepted Quantities” as long
as each inner bag contains 1
vial (20ml) and no more than
25 vials per outer package
(max 500ml)

Select transport method as established

Life Labs

MTS

Other local
medical courier

Canada Post
Pre-paid envelope,
pre-labeled with
Exempt Human Specimen

(pre-paid envelope available
thru online supply ordering
system for those without
another return option)

Clinic-collected swabs for HPV

Keep at room temp, place req +
swab in plastic envelope and place
one or more into outer envelope
labeled for delivery to CCSL



Deliver to
CCSL, BCCDC CPR
655 W12th Avenue
Vancouver, BC
V5Z 4R4